



RENOVODERM

(614) 453-5877 | 1330 Kinnear Rd. Columbus, OH 43212 | renovoderm.tech



1. Prepare the Wound Bed

Prepare the wound using standard methods to promote hemostasis and ensure it is free of debris and necrotic tissue.



2. Trim the Matrix (optional)

Trim PHOENIX Wound Matrix® (PHOENIX) to size such that the edges match the edges of the wound bed.



3. Smooth Out the Matrix

Gently smooth PHOENIX to remove all air bubbles and ensure direct contact of matrix with entire wound bed.



4. Anchor Matrix (optional)

Anchor PHOENIX with preferred method of fixation: staples, sutures, surgical glue, or reinforced adhesive skin closures (e.g., Steri-Strip™).



5. Rinse with Sterile Saline

Gently rinse PHOENIX and wound bed with sterile saline to facilitate full contact with the wound bed.



6. Non-Adherent Dressing

Apply an appropriate non-adherent dressing over PHOENIX to bolster it firmly in contact with the wound bed and to manage the wound environment.



7. Secondary Dressing

Select an appropriate secondary dressing to maintain adherence, protect the wound, and manage the wound environment.



8. Additional Application

Apply additional wrapping and bandages.